

## “Perfect fit Guarantee”

### Guidelines

- Please take note of the **IPR** marked in red on the lab slip you received back with your case.
- Deliver no more than 2 trays to patient at one time.
  - Evaluate progress before moving on to next tray.
  - Last tray should represent current anatomy.
- New trays should seat fully within 72 hours. Contact us if tray **does not** seat within this time period.
- For free course correction please inform Cottonwood Labs at or before reaching the halfway point in treatment.
- If the final outcome is not comparable to the finished working model you have received, please send us the working models (blue models with red wax), a new impression and a new Rx form and Cottonwood Labs will provide 1 refinement tray per arch at no cost to you.

Please refer to our tray section at [www.cottonwoodlabs.com](http://www.cottonwoodlabs.com) for additional information regarding our trays.

*“We'll make it Right”*